HOFFEN

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# Bathroom smart scale with analysis function

Model: SE-0992

## Table of Contents

1. INTRODUCTION.....	21
2. INTENDED USE.....	21
3. TECHNICAL DATA.....	22
4. PRINCIPLES OF SAFE USE.....	22
5. EXPLANATIONS OF THE SYMBOLS.....	24
6. THE CONTENTS OF THE SET.....	25
7. CONSTRUCTION.....	26
8. USE.....	27
9. CLEANING AND MAINTENANCE.....	36
10. PROBLEMS.....	36
11. STORAGE AND TRANSPORT.....	37
12. WARRANTY AND SERVICE.....	37
13. DECLARATION OF CONFORMITY WITH EUROPEAN STANDARDS.....	37
14. MANUFACTURER.....	37

## 1. INTRODUCTION

Thank you for choosing our product. We are confident that its use will be a great pleasure for you. Please read the user manual carefully before using the product, in particular the instructions on safe use. Please make it available to everyone who is going to use the device. Please keep this manual in order to be able to refer to it while using the product.

The colours of the product may slightly differ from the product shown in the illustrations in the manual and on the packaging.

## 2. INTENDED USE

This device is intended for private use and may not be used for commercial purposes.

The scale is used to measure your weight and analyze your body content, including body fat, water ratio, muscle and bone mass. To perform measurements, the device uses the bioelectrical method of body impedance (electrical resistance) analysis - BIA.

In order for the data to be reliable, the user enters the age, height and gender of the person to be analyzed. The device transmits very weak electrical impulses through the user's body, which pass through tissues and body fluids. The electricity meets the resistance of these tissues. The device, by measuring that resistance and using built-in algorithms, is able to analyze the body composition.

The device may show erroneous results for people with abnormal body structure, anatomical anomalies, bodybuilders, athletes, as well as dialyzed people, people suffering from osteoporosis, people with fever, as well as people taking drugs for the heart and cardiovascular system.

The measurements can be viewed using the mobile application for Android devices - the scale has a built-in Bluetooth module for communication with smartphones or tablets with Android system.

## 3. TECHNICAL DATA

Device: Bathroom smart scale with analysis function

Model: SE-0992

Lot number: **202009**

Measurement readings: LCD display / mobile application

Material: Plastic, glass, metal

Indicators: Battery charge status check, weight overload indicator

Automatic power on: Yes

Maximum weight: 180 kg

Accuracy: 0.1 kg

Measuring units Kg - kilogram, lb - pound, st - stone

Power supply: 4.5V = (3 x 1.5V = AAA battery)

## 4. PRINCIPLES OF SAFE USE

Read the entire user manual carefully before use. Keep it for future reference.

**WARNING !** This device may not be used by persons with cardiac pacemakers or other electronic medical devices. It cannot be used also by pregnant women and children under 10 years of age.

**CAUTION!** The device has a smooth surface and there is a risk of slipping and falling - a risk of serious injury. Before using the device, wipe the surface of the device and your feet dry. Stand on the scale to cover the metal parts with your feet so you won't tilt the device and risk falling.

1. Always abide by any kinds of prohibitions, regulations and warnings conveyed by the staff of the entity where you wish to use the device. Do not use the device near flammable fabrics.
2. This appliance may be used by children of at least 10 years of age and people with reduced physical and mental capabilities, or lack of experience and knowledge of the appliance, if supervised; or instructed about safe use so that the hazards involved are understood.

## HOFFEN

3. Children must not play with this device.
4. Unattended children must not clean or maintain the unit.
5. Cleaning and maintenance of the appliance must be carried out as described in section: 9. CLEANING AND CONSERVATION.
6. Avoid exposing the battery to very high/low temperatures (below 0°C/32°F and over 40°C/104°F). Extreme temperatures can affect service life of the batteries. Avoid contact with liquids and metal objects as it may cause partial or complete battery damage. Use the battery only according to its intended purpose. Do not destroy, impair or throw the battery into flames - it might be dangerous and may cause fire. Place worn or damaged batteries in a special container. Never open the battery. Dispose of the battery according to the instructions.
7. Avoid exposing the device to very high/low temperatures (below 0°C/32°F and over 40°C/104°F). Extreme temperatures can affect service life of the device.
8. The device should not be used in dusty conditions. Make sure that children and animals do not have access to the product. Damaged product can have cracks or broken parts with dangerous sharp edges. Do not expose the device to sunlight and do not place it in close proximity to heat-generating devices. Plastic parts may deform, which may affect the operation of the device or cause it to break down completely.
9. Protect the device from water and other liquids. Avoid using the device in environments with high humidity like kitchens or bathrooms. Water particles may cause moisture to appear in the device, which adversely affects the electronics and may cause damage to the equipment. Do not immerse the unit in water or other liquids.
10. The device has a glass surface, so do not place heavy objects on the device. Do not expose the device to situations in which something may fall on a glass panel and break it.
11. Do not use the unit if it has fallen to the ground or is visibly damaged.
12. The manufacturer is not responsible for any problems caused by unauthorized modifications to the equipment.

13. Only qualified personnel or an authorized service center may install or repair this product. Repairing by an unauthorized or unqualified service may cause device damage and warranty loss. The manufacturer is not responsible for problems caused by unauthorized modifications to the equipment made by the user.
14. The device emits electromagnetic waves in the radio frequency range from 2.402 GHz to 2.480 GHz, with a maximum power level of -4.63 dBm EIRP. The Bluetooth wireless speaker complies with safety standards and radio frequency recommendations. All wireless devices may cause interference to other devices.

**5. EXPLANATIONS OF THE SYMBOLS**

	<p>The device is marked with a crossed-out garbage bin, in accordance with the European Directive 2012/19/WE on used electrical and electronic Equipment (Waste Electrical and Electronic equipment - WEEE). Products marked with this symbol should not be disposed of or dumped with other household waste after a period of use. The user is obliged to get rid of used electrical and electronic equipment by delivering it to designated recycling point in which dangerous waste are being recycled. Collecting this type of waste in designated locations, and the actual process of their recovery contribute to the protection of natural resources. Proper disposal of used electrical and electronic equipment is beneficial to human health and environment. To obtain information on where and how to dispose of used electrical and electronic equipment in an environmentally friendly manner user should contact their local government office, the waste collection point, or point of sale, where the equipment was purchased.</p>
	<p>In accordance with directive 2013/56/UE on the disposal of batteries and accumulators, this product is marked with the symbol of a crossed-out waste bin. The symbol indicates that the batteries used in this product should not be disposed with regular household waste, but treated according to the legal directives and local regulations. It is not allowed to dispose of batteries and accumulators with unsorted municipal waste. Users of batteries and accumulators must use the available collection points of these items, which allows them to return, recycle and disposal. Within the EU, the collection and recycling of batteries and accumulators is a</p>

## HOFFEN

	subject to separate procedures. To learn more about existing procedures of recycling of batteries and accumulators please contact your local office or an institution for waste disposal or landfill.	
	The mark placed on the product indicates that the product complies with the basic requirements and other provisions of the so-called "Directive of the 'New Approach' Directive of the European Union.	
		The packaging of the device can be recycled in accordance with local regulations.
		Symbol for corrugated cardboard (packaging material).
	For indoor use.	

### 6. THE CONTENTS OF THE SET

After unpacking, check that all replaced parts are in the package and that the device is working properly. Included in the set:

- Bathroom smart scale with analysis function
- 3 x 1.5V = AAA battery
- User manual

*If any of the above items is missing or damaged, please contact your dealer.*

*We advise to keep the container in case of any complaints. Keep the container out of reach of children.*

You can obtain a free printed version of the manual with the large font size or download it from the web address below:

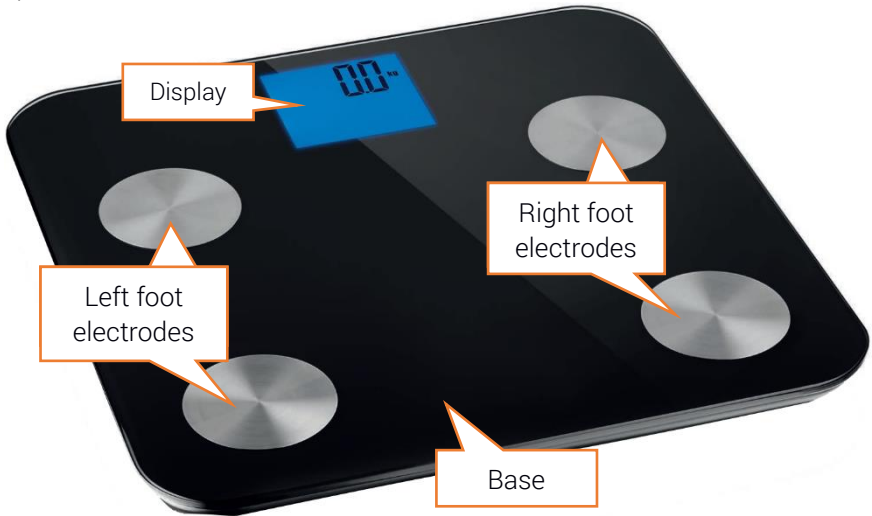
[www.mpotech.eu/qr/hoffen](http://www.mpotech.eu/qr/hoffen)

or by scanning the QR code.


APPEARANCE OF THE DEVICE AND ITS COMPONENTS

Upper part:

Lower part:

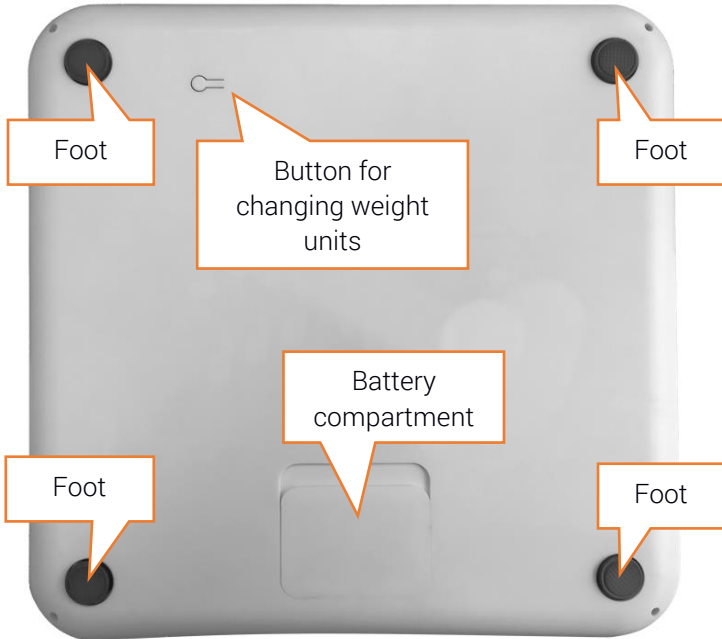
BEFORE THE FIRST START

 *All operations should be carried out gently so as not to damage the components.*

 *Set of 3 x 1.5 V 1.5V = AAA batteries included in the set.*

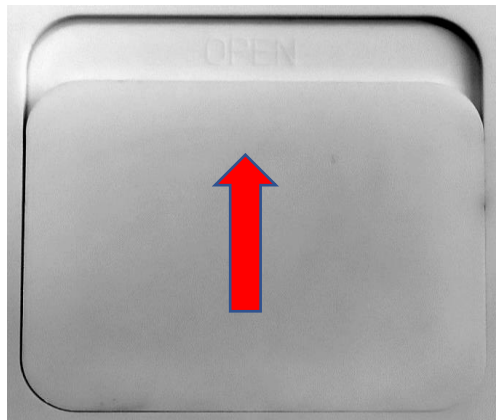Unpack the device. Check whether any of the components are damaged. Place the device with its lower part on a soft yet stable surface (carpet, bed, couch etc.) so that you have free access to the battery compartment.

Slide the battery compartment lid according to the arrow in the illustration.

Remove the cover and install the new batteries according to the polarity shown in the drawings inside the compartment. Replace and secure the battery compartment lid by sliding it in the opposite direction to opening. When replacing the batteries, follow the same procedure as when you first installed the battery, but remember to remove the entire set of used batteries and replace them with a new set of three batteries. Turn the scale over and place it on its feet in a place where it won't disturb any of the household members. Make sure that it is a stable, dry, flat and solid spot - tiles, panels, staves, etc. The device must not stand on any surface covered with material - a carpet, towel, etc. It's best if this place is well leveled. Adherence to the recommendations will ensure correctness of measurements

BASIC WEIGHT MEASUREMENT

 *Before using the device, ensure that your hands and feet are dry and that the surface of the device is clean and dry.*

*Stand on the scale with as little clothing as possible and do not hold on to any surrounding elements. Try not to wobble or move. Otherwise the measurement will not be accurate.*

*If you want to compare measurements, always try to do them in the same circumstances. We recommend that the weighing is performed in the morning.*

Stand with both feet on the properly prepared and positioned device. The device should be switched on automatically and the weight of the user should be measured. The measurement result should start blinking on the screen. After three seconds the result will stabilize and when it stops flashing we have a trustworthy measurement. When the user steps off the device, it automatically switches off after a few seconds.

### CONNECTING THE DEVICE TO A SMARTPHONE OR TABLET AND CONFIGURATION OF THE APPLICATION

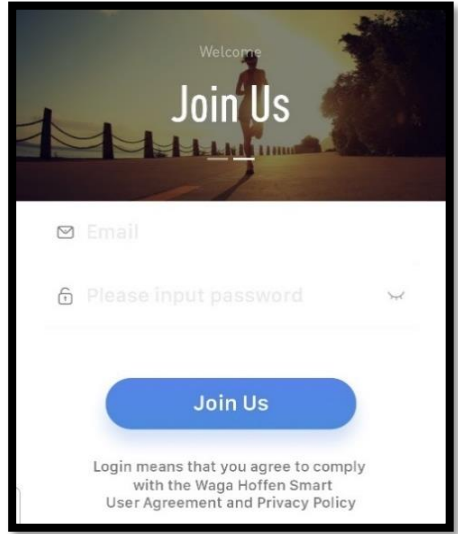
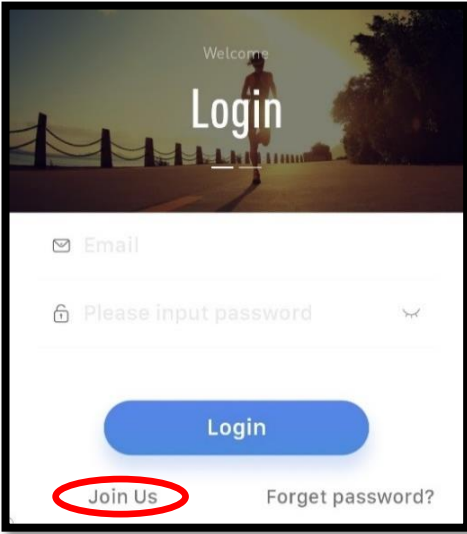
*The application for operating the device works on smartphones and tablets using the Android operating system (version 5.0 to higher) and with an active Bluetooth module.*


To use the advanced functions of the device, download the **HOFFEN SMART SCALE** application to your phone or smartphone using the address:

<https://play.google.com/store/apps/details?id=com.chipsea.btcontrol.pl>  
or by scanning the QR code above

Subsequently install the app by giving it required permissions. Switch on Bluetooth on your smartphone or tablet. Start the application and assign the appropriate permissions. During the first start, select the "**Join Us**" option by swiping the screen. The application will ask you to create an account. Enter your email address and password and confirm it by pressing "**Join Us**".


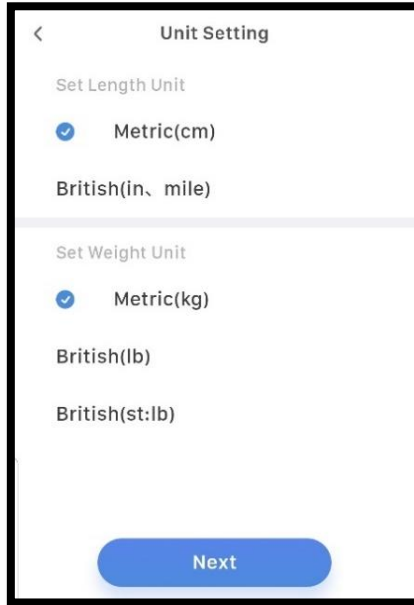
Then set the units of length and weight - by default one meter and one kilogram is set.


# HOFFEN

Then you can set the photo, name, gender, date of birth, height and then the target.

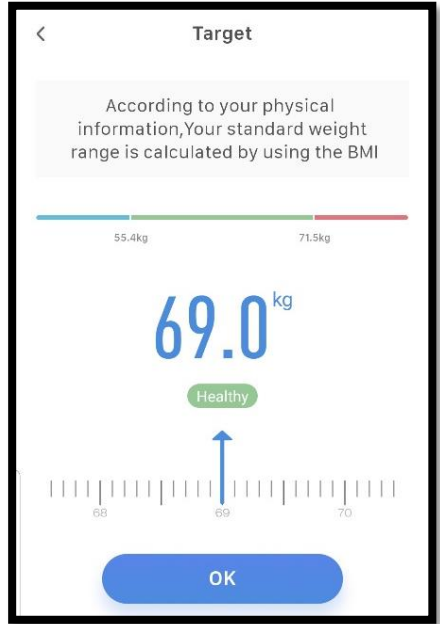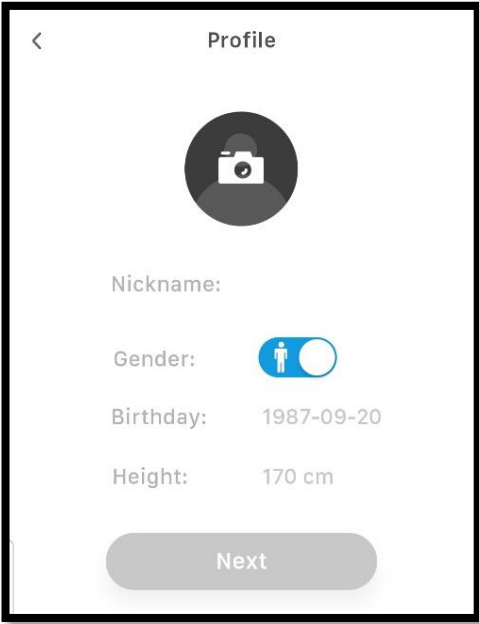The main screen will appear.
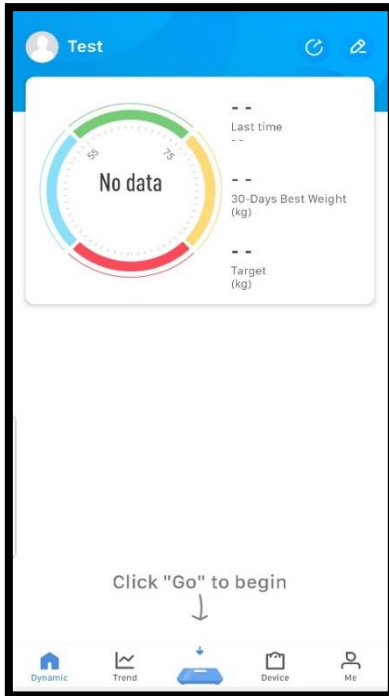
# HOFFEN

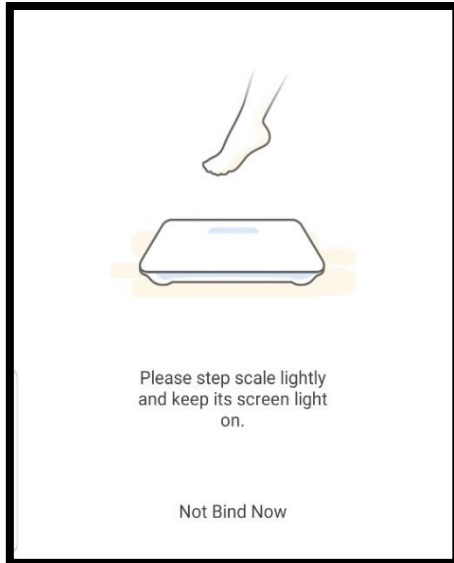
Select "Go" to begin and pair the device with the application.

The screen as below will appear. Select "Add device"

Follow the instructions from the application.


The scale will automatically connect to the application.

# HOFFEN
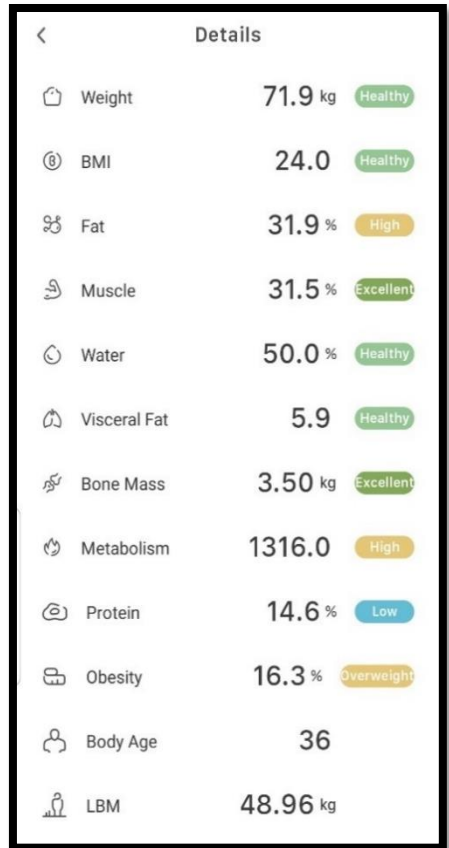
## ADVANCED MEASUREMENT OF BODY COMPOSITION PARAMETERS

The device can perform advanced body composition measurements. To achieve that one has to stand on the scale with bare feet to cover the electrodes on the surface panel (two for each foot). The device will measure the user's body weight first and then analyse the body composition - this will be indicated by an animation of the batteries on the screen of the device. Application will show a readings similar to the one presented below. The measurement will be stored in the application memory


The application offers the presentation of such parameters as weight, BMI (Body Mass Index - weight to growth ratio), percentage of body fat, muscles, water, and visceral body fat (visceral fat, that is, the one found on internal organs, not directly under the skin), bone mass, and resting metabolism (the minimum amount of calories to be delivered to the body per day). Additionally, during subsequent measurements the difference in weight will appear in relation to the previous (last) measurement.

## HOFFEN

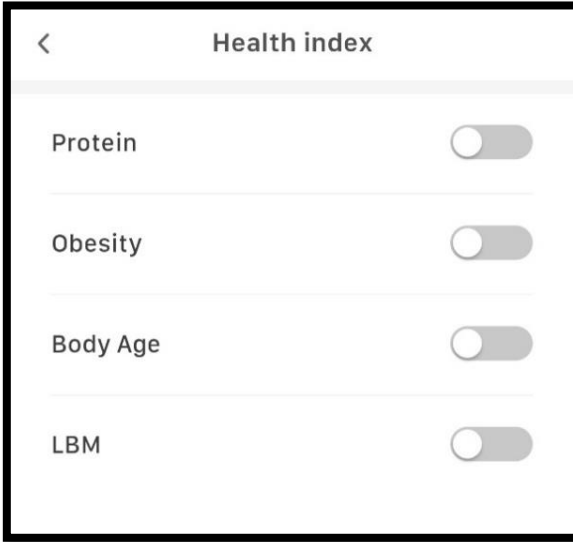In addition, you can enable the presentation of additional parameters called **Health Index** (Settings > Health Index). These are the amount of protein in the body, obesity level (difference between the actual and ideal weight), biological age (presents the condition of the body calculated on the basis of measurements made), LBM (Lean Body Mass - body weight without body fat).

### VIEW MEASUREMENT HISTORY AND GRAPHS

The application also has the ability to present data from measurements on charts. In order to view them, you need to select the **"Trend"** tab at the bottom of the page. The graphs will appear after the measurements have been taken on at least two consecutive days.

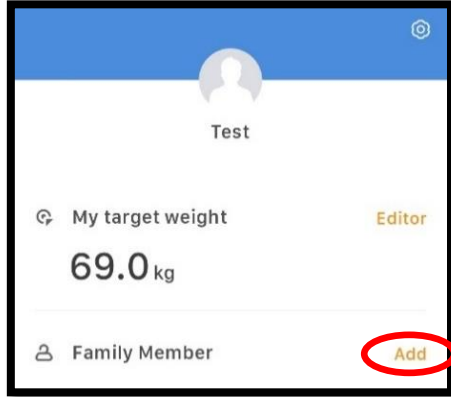
One can look at the top for parameters such as Weight, BMI, Fat, Muscles, Water, Visceral, Bone Mass, BMR. If you don't see all of them, you have to swipe them to the left to get more parameters.

## ADDING AND DELETING USERS

By selecting the **"Me"** tab at the bottom you can edit your data and add new users to your device.


## MEASUREMENT IN "FAMILY MEMBER" MODE

The scale can also be used in **"Family member"** mode.

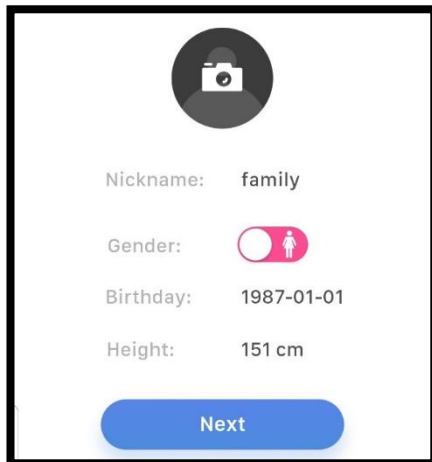
The measurement data (gender, date of birth, height) must be provided each time for the device to be able to perform the correct measurements. The data will not be saved in the application database.

## HOFFEN


### RECOVERY OF FORGOTTEN ACCOUNT PASSWORD

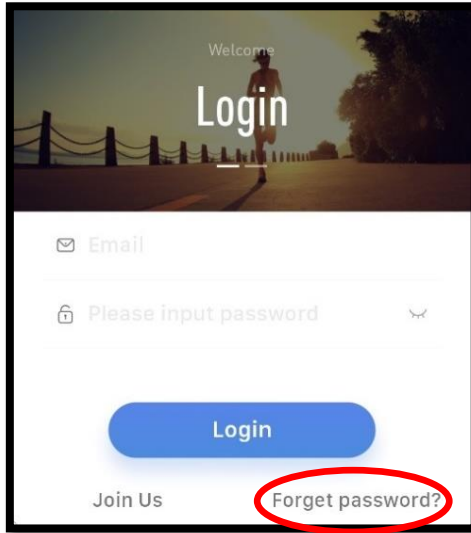If your account password has been forgotten, you can recover it. To do this, start the application and select the "Login" option. Then select the "**Forgot password?**" link at the bottom right.

Enter the e-mail address you used to create your user account. Enter the new password in the "**New password**" field. Press the "**Obtain VCode**" button. You have 180 seconds to read the code sent to the given address, type into in the "**Verify code**" field and confirm it.

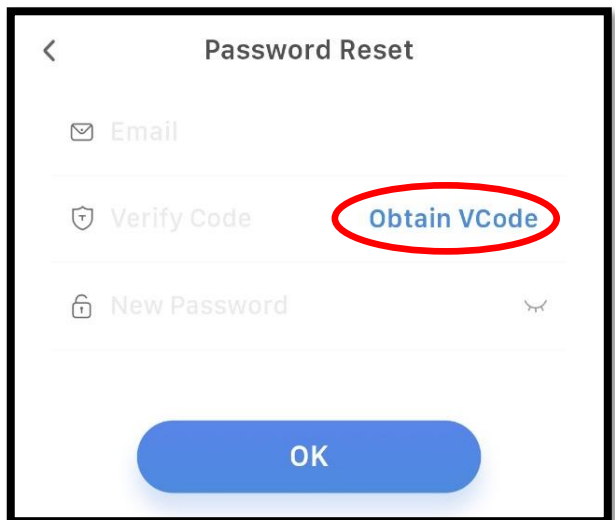Proper verification will result in a new password being in force.


## CHANGE WEIGHT UNITS

The device is able to perform measurements in different units. It takes measurements in **kg** by default.

Using the UNIT button at the bottom, the user can switch the device to measurements in units such as lb - pounds, and st - stones.

For measures other than kilograms, the height in feet and inches - the units of measurement used in the United Kingdom - must also be given. If you change units in

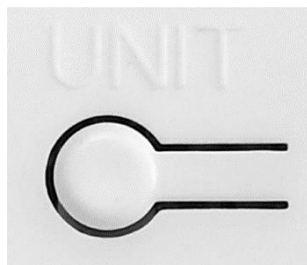weight, but not in the application, the application will convert the measured values to the units set in the application and store such results in the memory.

## 9. CLEANING AND MAINTENANCE

### CLEANING OF THE RIGID PARTS OF THE APPLIANCE

 *Cleaning should only be carried out when the device is switched off and cooled down, the mains plug is disconnected from the mains socket.*

The device may only be cleaned with a dry cloth or paper towel. Do not immerse the device in water. To clean and disinfect the electrodes and the surface, use a cloth lightly sprayed with alcohol.

It is forbidden to use water or any other liquids to clean the device.

## 10. PROBLEMS

In case of any problems with the device, please refer to the following tips.

A message appears on screen: <b>Lo</b>	Replace the battery pack as the current is already discharged.
A message appears on screen: <b>Err</b>	Message indicates a measurement problem Re-measure accurately stepping and standing still on the scale. It also appears when a person weighs more than 180kg.
A message appears on screen: <b>Err2</b>	Message indicates a problem with the measurement of the body's composition. Re-measure precisely by standing on the scale and covering with bare feet the electrodes designed for the left and right foot. In some cases, it helps to gently moisturise the soles of the feet with water.
A message appears on screen: <b>FATL</b>	This message appears when the measurement shows that the percentage of body fat is less than 5%.
A message appears on screen: <b>FATH</b>	This message appears when the measurement shows that the percentage of body fat is higher than 50%.
I cannot switch the device on	Check that the battery is inserted correctly - that the polarity is maintained.

## HOFFEN

Weight measurements are strange or erroneous.	Place the scale on a hard, flat and level surface.
Body composition measurements are odd or varied, despite measurements made every few minutes.	Stand on clean electrodes with bare feet. Soles of the feet covered in calluses or even body lotion can cause the results to be erroneous. Measuring before and after a meal or drinking fluids, bowel movements, urination will vary.
Forgot my password to log in	See the chapter <b>RECOVERY OF FORGOTTEN ACCOUNT PASSWORD</b>
<b>If the problem persists, contact the manufacturer's service department.</b>	

### 11. STORAGE AND TRANSPORT

#### STORAGE OF EQUIPMENT

If you do not use the device for a long period of time, after turning it off, cleaning and removing the battery pack, store it in a dry place away from heat and moisture sources. It is best to place the device in its original packaging, away from children and pets.

#### TRANSPORT OF THE DEVICE

It is recommended to transport the device in its original packaging. Do not expose the unit to excessive vibration or shock during transportation. When handling, only carry the unit after it has been switched off, holding it securely by the housing. Do not expose the device to heat or moisture.

### 12. WARRANTY AND SERVICE

The product is covered by a 2 year warranty from the date of sale of the product. Batteries are not covered by the warranty - it is a consumable material. If you need warranty repair, please contact the manufacturer's hotline. The product returned for repair should be complete and preferably in its original packaging.

### 13. DECLARATION OF CONFORMITY WITH EUROPEAN STANDARDS

 mPTech Sp. z o.o. hereby declares that the device complies with Directive 2014/53/EU. The full text of the EU Declaration of Conformity is available at the following Internet address:

[www.mptech.eu/qr/hoffen](http://www.mptech.eu/qr/hoffen)

### 14. MANUFACTURER

mPTech Sp. z o.o.  
Nowogrodzka 31 street  
00-511 Warsaw  
Poland

Helpline: (+48 71) 71 77 400  
Manufactured in China  
Lot number: 202009