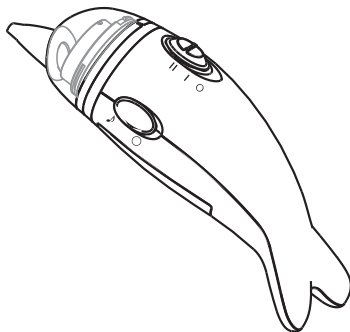
HOFFEN

ASPIRATOR DO NOSA | NASAL ASPIRATOR

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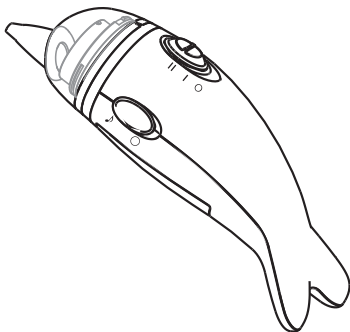
 Guangzhou Berrcom Medical Device Co.,Ltd,
No.38 Huanzhen Xi Road, Dagang Town,
Nansha, Guangzhou, China

 Shanghai International Holding Corp. GmbH (Europe)
Eiffestrasse 80, 20537 Hamburg, Germany

 182/21

NASAL ASPIRATOR
MODEL: NC005

MANUAL

Actual colours may vary slightly from those illustrated.

CONTENTS

1. INTENDED USE AND APPLICABLE PEOPLE	17
2. SAFETY INSTRUCTION	17
3. PRODUCT FEATURES	19
4. PRODUCT DESCRIPTION	20
5. TECHNICAL FEATURES	20
6. BATTERY INSTALLATION	21
7. UNIT ASSEMBLY	21
8. TROUBLESHOOTING	23
9. MAINTENANCE	23
10. GUIDELINES	23
11. EXPLANATION OF SYMBOLS	24
12. EMC DECLARATION	25
13. AUTHORIZED REPRESENTATIVE OF THE MANUFACTURER	29

INTRODUCTION

Dear customer,
thank you for purchasing this electronic baby nasal aspirator.
We hope it will be useful for you. And recommend that read the use instruction carefully before cleaning your baby's nose.
This device may be used for professional purposes or for personal home use.

1. INTENDED USE AND APPLICABLE PEOPLE

Intended Use: Quickly and gently removes excess mucus for alleviation infection of upper respiratory tract.

Applicable People: Children (including infant).

2. SAFETY INSTRUCTION

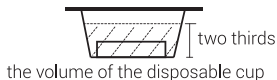
NOTE: The user must check that the equipment functions safely and see that it is in proper working condition before being used.

1. Do not allow children to play with this device because it is not a toy. Keep the device out of children's reach to prevent small parts and accessories from being swallowed by them accidentally.
2. This device is to clean stuffy noses, do not use it in baby's mouth or ears.
3. Do not insert the silicone suction nozzle into the nostril too deeply.
4. Do not use it on the baby for a long time, a few seconds will be enough.
5. Do not use this device if there is any wound, symptom of inflammation, or bleeding at the nose.
6. In the case of infection, fever, nasal irritation, or hemorrhage, please stop using this device and consult your physician.
7. Do use the accessories provided by manufacturer.
8. Do use scatter or throw this device; otherwise, it would damage the device.
9. Do use alkaline battery but not the rechargeable ones.
10. Do move the battery out when it is not used for an extend period of time and follow the instruction for battery installation.
11. Replace the batteries when the suction power becomes weak during use.
12. Keep the use manual well.
13. Please note that the product's operation mode is intermittent.

It is recommended that it lasts at most 10 minutes after the operation, it should stop using after 5 minutes. Its intended use is to clean the baby nose's liquid and nasal secretions.

ATTENTIONS

- When using the machine, make sure the infant opens the mouth to keep the ventilation of air inside his nose in case of the occurrence of negative air pressure inside baby's head and unease.
- Clean the machine after use.
- Don't dip the whole machine into water.
- Sterilize the components including nozzle and transparent hood to prevent the cross infection.
- The machine is not equipped with anti counter current device, the user must install the disposable cup when the machine is working in case the mucus is sucked into the inside of the machine and cause damage.
- Change the disposable cup or clear off the mucus inside the disposable cup when the mucus reaches two thirds of the cup's volume.

- When the machine is working or there is already mucus inside the disposable cup, don't put the machine upside down otherwise the mucus will enter the engine of the machine and cause damage.

right (✓)

wrong (✗)

- The user must dry the component as the picture shows after the sterilizes it with boiled water or steam in case the water inside the component was sucked into the engine of the machine.

- Please remove the batteries when the device will be unused for more than 14 days.
- When user re-installs the seal ring, he must firstly clear off the dust where the seal ring will be installed and make sure that the complete body of the seal ring is attached to the machine evenly. If not, the suck power will be influenced.

Silicone seal ring

Put the seal ring evenly where it should be installed of the machine.

Precautions before using

1. Do not forget to sterilize the accessories before and after each use, including the probes.
2. Do wash your hands before using the accessories.
3. Do check the waterproof seal regular in time.
4. Do not place the whole device in any liquid.

Advice to parents

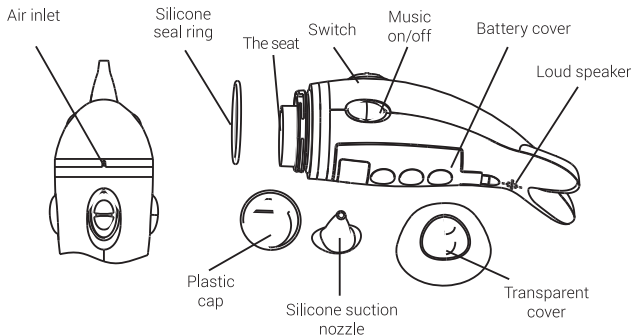
1. Do clean your hands before taking care of your children.
2. Do uplift your baby's head during operation and make sure smooth air draft.
3. Do moisture the nasal secretion before operation.
4. Do make sure the baby's mouth is open during operation, so that air will flow smoothly in nostril.
5. Warm Prompt to Avoid Infection
 - A. Do not kiss sick baby
 - B. Do not kiss a baby when you are sick
 - C. Do isolate the sick baby.
 - D. Do not cross use personal bacteria-bearing stuffs to baby. Should you have any further problems, please contact your retailer.
6. After each use, clean and sterilize parts including nozzles and transparent hoods with 75% medical alcohol to prevent cross-infection.

3. PRODUCT FEATURES

1. Two options to adjust the suction power.
2. With sweet music function (optional).
3. Convenient hygiene and secure suction cups.
4. Keep the babies' nasal cavity clean easily.

Since the babies no more than one year old can only breathe by nose, it is essential to clean their nasal secretion and mucus regularly in case of illness


4. PRODUCT DESCRIPTION

Accessories

- 1 pc Silicone suction nozzle
- 15 pcs Plastic caps
- 2 pcs AA alkaline batteries
- 1 pc Manual
- 1 pc Spared silicone seal ring

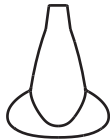Silicone suction nozzle

Silicone seal ring

Plastic caps

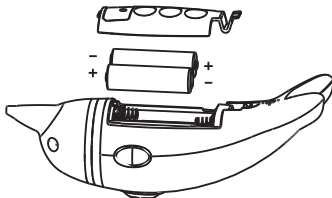
Please contact Joinco Polska Sp. z o.o. should you want to purchase any of these elements.

5. TECHNICAL FEATURES

- Normal using condition
 - Ambient temperature: 10°C ~ 40°C (50°F ~ 104°F)
 - Relative humidity: ≤85%
 - Pressure altitude: 700 hPa ~ 1060 hPa
- Storage and shipping condition
 - Ambient temperature: -20°C ~ 55°C (-4°F ~ 131°F)
 - Relative humidity: ≤95%
 - Pressure altitude: 700 hPa ~ 1060 hPa

- Batteries: 3 V (2 pcs AA batteries)
- Product size: 172 x 54 x 72 mm (L x W x H)
- Product weight: about 151.5 g (without batteries)
- Maximum suction: 58 kPa

6. BATTERY INSTALLATION

- Open by pulling the battery compartment in the lower part of the device.
- Put a couple of AA batteries in, and make sure the polarity is correct.
- Do not use chargeable battery. And do not mix the new battery with an old one.
- Close the battery compartment.

7. UNIT ASSEMBLY

1.

- After insert the batteries ① (When using for the first time), it is to place a hygienic cup ④ into the sealing base ③.
- Make sure the silicon seal is properly positioned ②, allowing the nasal aspirator being sealing up well.
- Put the cover ⑤ in right place, and turn the cover till it is sealed up.
- Install the silicone suction nozzle ⑥.

2. Toggle switch to the I or II to select the size of the suction.
3. Put the probe into the baby's nasal cavity carefully (please note that the head needed to be uplifted slightly), then can clean the nasal cavity efficiently.
4. Press the switch on "O" location after using, then the nasal aspirator will be shut down.
5. During operation, please press the music button on "♪" location, the music is on, when you press it on "O" location, the music will be off.
 - Make sure the baby's mouth is open during operation to make air draft in nostril cavity.
 - Do not suck the nostril wall.
 - Slow down the movement and no harsh and sudden, in case of any damage to the baby.
 - Change the hygienic cup when it is necessary.

 Attention: When using the physical saline water, the mixed solution with nasal secretion is probably drained into bronchia. Do make sure such mixed solution is sucked away by nasal aspirator.

 Attention: Do make sure child's mouth is open during operation to make draft in nostrils.

 Attention: Dispose the hygienic disposable cup when used up and clean the main unit with proper detergent.

 Attention: Do not dig the device into any liquid.

 Attention: Do sterilize every a period of time.

In order to ensure correct safety and operating conditions, batteries should always be installed and replaced by an adult.

1. After batteries are depleted, remove them from the toy and replace with new ones.
2. It is prohibited to use new and depleted or different types of batteries at the same time, i.e. standard (e.g. carbon-zinc) or rechargeable (e.g. nickel-cadmium) batteries.
3. Never allow the opposite power supply poles to contact each other. Insert batteries observing the correct polarization. Incorrectly used batteries may explode or leak.
4. Never place batteries near heat sources, do not put them in fire or leave in a place exposed to sunlight.
5. Use only batteries of the same type or equivalent.
6. Replace batteries when damaged or leaking. Do not disassemble the batteries.
7. Immediately wash your skin and clothes after contact with the electrolyte from a leaking battery.
8. Remove batteries from the toy if it will not be used for longer time.
9. Do not put used batteries into the household waste basket. Dispose of the used batteries to the appropriate recycling facility.

10. Batteries are not suitable for recharging, do not attempt to recharge them.
11. Do not recharge the rechargeable batteries when hot. Before recharging remove the rechargeable batteries from the toy.
12. The rechargeable batteries should be recharged under the adult supervision.
13. For longer rechargeable battery life recharge them regularly. Use the same or equivalent battery charger as recommended.
14. Before handing the toy to your child make sure the batteries are safely locked in the battery compartment. Do not allow your child to play with the toy, if the batteries are not securely locked. Batteries are small objects which can be swallowed by a child.
15. Do not short circuit the battery contacts.

8. TROUBLESHOOTING

Should any of the problems incur, please refer to the troubleshooting.

Troubles	Solutions
No aspiration property.	<ol style="list-style-type: none"> 1. Check the sealing joint. 2. Check the hygienic cove in right place or not. 3. Check the silicon probe in right place or not
No reaction with switch ON/OFF button.	Check the battery polarity.

9. MAINTENANCE

1. Clean the device exterior with proper detergent (e.g. alcohol).
2. Tear down the moveable compartment and sterilize with boiled water or steam, but not in the microwave.
3. Clean the device with dry and soft cloth after sterilization.
4. Do not expose the nasal aspirator to sunshine and damp area directly.
5. Do not use erosive chemicals to clean the device.

Attention: Extract battery when the device is no used for an extend period of time.

10. GUIDELINES

This device complies with the EU Directive 93/42/EEC concerning medical products, and the European Standard EN60601-1-2 and is subject to particular precautions with regard to electromagnetic compatibility.

11. EXPLANATION OF SYMBOLS

IEC 60417-5333, type BF applied part.

IP22

Ingress protection provided by enclosures.

2 – protection against foreign solid bodies with a diameter of 12.5 mm and greater.

2 – dripping water when tilted at 15°.

Refer to instruction manual / booklet.

This symbol means that the product must not be disposed of with household (mixed) waste. Polish regulations prohibit the disposal of WEEE (waste electrical and electronic equipment) with mixed household waste, and offenders will be fined.

You can help prevent any negative environmental and health impact of WEEE components by properly disposing of your WEEE.

This symbol shall be accompanied by the name and the address of the manufacturer.

Designation of container to which the packaging should be discarded – plastic/metal.

Packaging elements are to be disposed into appropriate containers.

Designation of the packaging material – corrugated cardboard.

Designation of the packaging material – corrugated cardboard.

Disposal of batteries.

Discard spent and completely used batteries in a waste bin marked as approved for this type of waste or dispose of them at a special waste collection centre or with a seller of electrical products.

Pb is the chemical symbol of lead and indicates that each battery

EC REP

Authorised representative's data.

LOT

Lot number designation.

When the "Green Dot" restriction appears on packaging, it signifies that the importer has made a financial contribution towards development and operation of the national packaging recovery and recycling system in accordance with the standards resulting from the laws of Poland and the European Union on packaging and packaging waste.

12. EMC DECLARATION

Guidance and manufacturer's declaration – electromagnetic emission – for all EQUIPMENT AND SYSTEMS

Guidance and manufacturer's declaration – electromagnetic emission		
The NC005 Nasal Aspirator is intended for use in the electromagnetic environment specified below. The customer or the user of NC005 Nasal Aspirator should assure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The NC005 Nasal Aspirator uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The NC005 Nasal Aspirator is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Not applicable	
Voltage fluctuations flicker emissions IEC 61000-3-3	Not applicable	

Guidance and manufacturer's declaration – electromagnetic immunity – for all EQUIPMENT and SYSTEMS

Guidance and manufacturer's declaration – electromagnetic immunity			
The NC005 Nasal Aspirator is intended for use in the electromagnetic environment specified below. The customer or the user of the NC005 Nasal Aspirator should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±8 kV contact ±2 kV, ± 4 kV, ± 8 kV, ±15 kV air	±8 kV contact ±2 kV, ± 4 kV, ±8 kV, ±15 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrostatic transient / burst IEC 61000-4-4	±2 kV for power supply lines ±1 kV for input/output lines	Not applicable	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	±1 kV differential mode ±2 kV common mode	applicable	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	0% UT; 0,5 cycle g) At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° 0% UT; 1 cycle and 70% UT; 25/30 cycles Single phase at 0° 0% UT; 250/300 cycle	applicable	Mains power quality should be that of a typical commercial or hospital environment. If the user of the NC005 Nasal Aspirator requires continued operation during power mains interruptions, it is recommended that the NC005 Nasal Aspirator be powered from an uninterruptible power supply or a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	30 A/m	30 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
NOTE : UT is the a. c. mains voltage prior to application of the test level.			

Guidance and manufacturer's declaration – electromagnetic immunity – for EQUIPMENT and SYSTEM

Guidance and manufacturer's declaration – electromagnetic immunity		
The NC005 Nasal Aspirator is intended for use in the electromagnetic environment specified below. The customer or the user of the NC005 Nasal Aspirator should assure that it is used in such an environment.		
Immunity test IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
		<p>Portable and mobile RF communications equipment should be used no closer to any part of the NC005 Nasal Aspirator, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended separation distance</p>
<p>Conducted RF IEC 61000-4-6</p> <p>Radiated RF IEC 61000-4-3 3 Vrms 150 kHz to 80 MHz 6 V in ISM and amateur radio bands between 0,15 MHz to 80 MHz 10 V/m 80 MHz to 2,7 GHz 385MHz-5785MHz Test specifications for ENCLOSURE PORT IMMUNITY to RF wireless communication equipment (Refer to table 9 of IEC 60601-1-2:2014).</p>	<p>Not applicable</p> <p>150 kHz do 80 MHz</p> <p>6 V w ISM i amatorskich pasmach radiowych pomiedzy 0,15 MHz a 80 MHz</p> <p>10 V/m</p> <p>80 MHz to 2,7 GHz</p> <p>385MHz-5785MHz Test specifications for ENCLOSURE PORT IMMUNITY to RF wireless communication equipment (Refer to table 9 of IEC 60601-1-2:2014).</p>	<p>$d = \left[\frac{3,5}{F_1} \right] \sqrt{P}$</p> <p>$d = \left[\frac{12}{F_2} \right] \sqrt{P}$</p> <p>$d = \left[\frac{3,5}{E_1} \right] \sqrt{P}$ 80 MHz do 800 MHz</p> <p>$d = \left[\frac{7}{E_1} \right] \sqrt{P}$ 800 MHz do 2,7 GHz</p> <p>where p is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, a should be less than the compliance level in each frequency range.</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol:</p> 
<p>NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.</p> <p>NOTE 2 These guidelines may not apply in all situations. Electromagnetic is affected by absorption and reflection from structures, objects and people.</p>		

- a The ISM (industrial, scientific and medical) bands between 150 kHz and 80 MHz are 6,765 MHz to 6,795 MHz; 13,553 MHz to 13,567 MHz; 26,957 MHz to 27,283 MHz; and 40,66 MHz to 40, 70 MHz. The amateur radio bands between 0,15 MHz and 80 MHz are 1,8 MHz to 2,0 MHz, 3,5 MHz to 4,0 MHz, 5,3 MHz to 5,4 MHz, 7 MHz to 7,3 MHz, 10,1 MHz to 10,15 MHz, 14 MHz to 14,2 MHz, 18,07 MHz to 18,17 MHz, 21,0 MHz to 21,4 MHz, 24,89 MHz to 24,99 MHz, 28,0 MHz to 29,7 MHz and 50,0 MHz to 54,0 MHz.
- b Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the NC005 Nasal Aspirator is used exceeds the applicable RF compliance level above, the NC005 Nasal Aspirator should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the NC005 Nasal Aspirator.
- c Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

Recommended separation distances between portable and mobile RF communications equipment and the EQUIPMENT or SYSTEM – for EQUIPMENT and SYSTEMS

Recommended separation distances between portable and mobile RF communications equipment and the NC005 Nasal Aspirator				
The NC005 Nasal Aspirator is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the NC005 Nasal Aspirator can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the NC005 Nasal Aspirator as recommended below, according to the maximum output power of the communications equipment.				
Rated maximum output of transmitter W	Separation distance according to frequency of transmitter m			
	150 kHz to 80 MHz outside ISM and amateur radio bands $d = \left[\frac{3,5}{V_1} \right] \sqrt{P}$	150 kHz to 80 MHz in ISM and amateur radio bands $d = \left[\frac{12}{V_2} \right] \sqrt{P}$	80 MHz to 800 MHz $d = \left[\frac{3,5}{E_1} \right] \sqrt{P}$	800 MHz to 2,7 GHz $d = \left[\frac{7}{E_1} \right] \sqrt{P}$
0,01	0,12	0,20	0,035	0,07
0,1	0,38	0,63	0,11	0,22
1	1,2	2,00	0,35	0,70
10	3,8	6,32	1,10	2,21
100	12	20,00	35	70
For transmitters rated at a maximum output power not listed above the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.				
NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.				
NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.				

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